
Trust our experience

Because Sculptra® is brought to you by the Medical Aesthetics division of sanofi-aventis, one of the world's leading pharmaceutical companies, you can confidently trust its science and its safety profile.

To find out more talk to your Medical Aesthetics practitioner, visit www.sculptra.co.uk or call 0800 085 2046.

For full Product Information, consult your Medical Aesthetics practitioner and/or refer to the product leaflet. sanofi-aventis. One Onslow Street, Guildford, Surrey GUI 4YS. Date of preparation: May 2010. GB. POL 10.04.06.

sanofi aventis

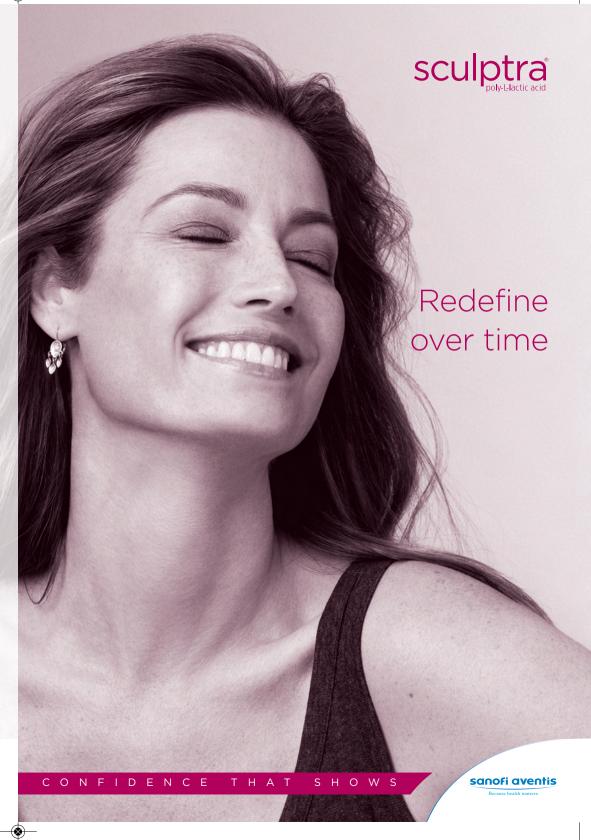



Facing up to ageing

Ageing is a fact of life. And it usually shows most on our faces. Often we begin to notice its effects in our 20s when fine wrinkles, light blemishes and sunspots begin to appear. But why shouldn't the way we still feel on the inside continue to be subtly reflected on the outside?

You know what you want

With the range of modern, non-surgical aesthetic treatments now available to help regain what our skin has lost, more of us are choosing to help lessen the impact of ageing and cleverly refresh the way we look.

What is suitable for you?

While treatment selection and results depend on your age, type of lines, the particular substance being injected and what you wish to achieve, your practitioner will discuss and advise on what's best for you.

But if you are looking for a gradual, longer-lasting result, there is one tried and trusted treatment that may be just right for you.

Redefine over time



Collagen is the substance your body produces to give your skin a firm and youthful appearance. But from an early age the amount you produce slows down and facial volume is lost. Sculptra® can redress the balance and help you achieve the natural look you want, to reflect how you feel.

How it works

Sculptra® works differently from traditional fillers. It's a unique collagen replenishing treatment that helps to gradually and subtly reduce lines, wrinkles and folds. By helping to replenish your own facial collagen and restore lost volume, it can give you a fuller, more natural look that can last for up to 25 months.

Suitable areas for Sculptra®

Sculptra® treats most areas of your face that may be

concerning you, particularly deeper lines, wrinkles and folds.

On average, 3 treatments are required and results should be noticeable after approximately 6 weeks (however, in some patients this may take longer or require more treatments).



Your practitioner will advise on what is right for you.







Confidence that shows

"Friends say how well I look" A.B.

> "I truly feel that I have got my face back" C.K.

"I have people saying to me that my skin looks fresher"

L.B.

Before-to-after with Sculptra®

Name: Elissa (aged 48)



Session 2 results





Total vials and sessions: 5 vials over 3 sessions

Name: Franca (aged 40)







Total vials and sessions: 6 vials over 4 sessions



Session 4 results





Name: Renata (aged 53)

Session 2 results

Month 20 after first injection results



Total vials and sessions: 5 vials over 4 sessions

Individual results and treatment regimens may vary. (All photos unretouched)

With Sculptra® 95% of patients remained satisfied with the results







Reassuring from the start

Who will perform my procedure?

A qualified Medical Aesthetics practitioner who has completed the Sculptra® training programmme certified by sanofi-aventis.

For a list of Medical Aesthetics practitioners visit www.sculptra.co.uk or call 0800 085 2046.

Does treatment hurt?

As with all injections you will probably feel some discomfort. You may even notice a brief burning sensation at the injection site.

Applying an ice pack after treatment will help calm your skin. If necessary your Medical Aesthetics practitioner will also give you a local anaesthetic.

What about afterwards?

Your Medical Aesthetics practitioner will give you special aftercare instructions.

For optimum effects it is important to massage the treatment area twice daily for one month.

Inform your Medical Aesthetics practitioner if any side effects occur, for example inflammation or small bumps in the treated area.

What about side effects?

The most common side effects with the use of Sculptra® include bleeding, tenderness or pain, redness, bruising or swelling at the site of the injection. These side effects generally resolve in 2-6 days. One possible delayed side effect can be small bumps under the skin in the treated area. These small bumps may not be visible, and you may notice them only when you press on the treated skin. As with all procedures that involve an injection through the skin, there is a risk of infection.

Any side effects should be reported to your Medical Aesthetics practitioner.

Side effects can be avoided or reduced if you ensure you have treatment by someone who has completed the Sculptra® training programme and you follow your proper aftercare tips.

Results with Sculptra® vary between individuals, depending upon a number of factors and, similar to other medical treatments, results cannot be guaranteed. Any advice required, before or after treatment, should be requested from your Medical Aesthetics practitioner and not the company who manufacture the product.



